Link do produktu: <https://higienadent.com/premium-plus-koncowka-do-skalera-e1e2-emswoodpecker-1szt-p-3296.html>

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
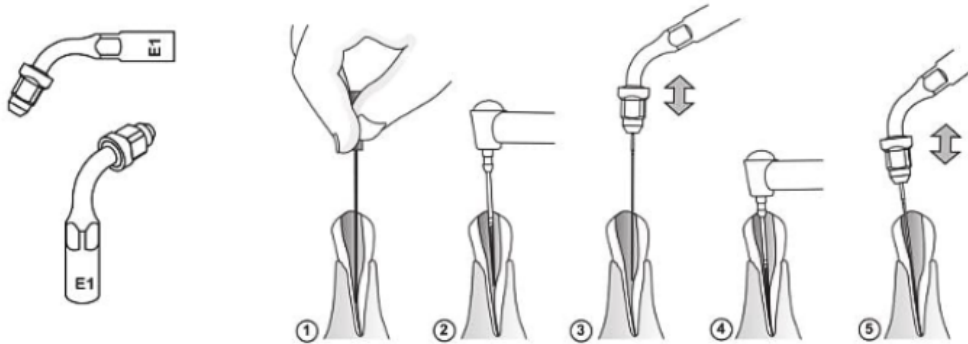
E1 (ED1/ES1)

120° file holder used to hold the file diameter $\Phi 0.7\text{mm}$ or $\Phi 0.8\text{mm}$. Recommended for irrigating of root canal, usually used for anterior teeth.


Use only the provided wrench (TW-E/TW-A) to tighten the file holder to handpiece, as well as tighten the file to the file holder. Make sure not over tighten it, and don't tighten the holder nut without file in the hold. Don't mix-install $\Phi 0.7\text{mm}$ and $\Phi 0.8\text{mm}$ file on the file holder, the $\Phi 0.8\text{mm}$ file will not be able to be installed after a $\Phi 0.7\text{mm}$ installed.

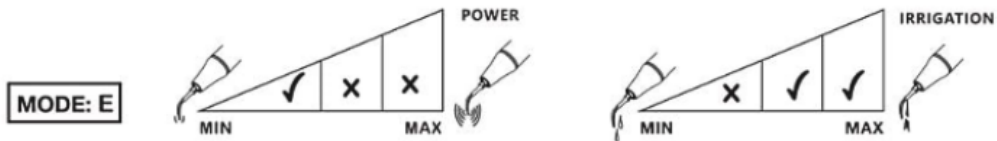
E2(ED2/ES2)

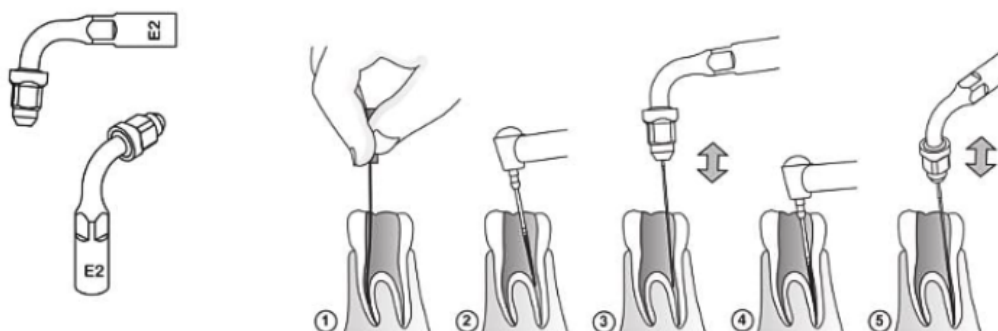95° file holder used to hold the file diameter $\Phi 0.7\text{mm}$ or $\Phi 0.8\text{mm}$. Recommended for irrigating of root canal, usually used for molar teeth.

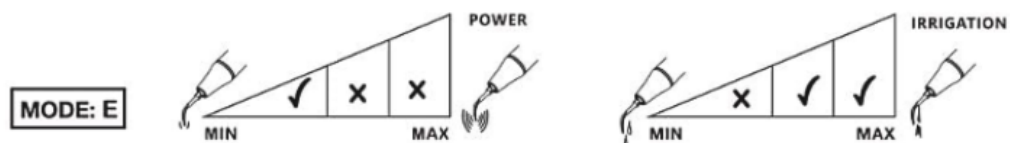Use only the provided wrench (TW-E/TW-A) to tighten the file holder to handpiece, as well as tighten the file to the file holder. Make sure not over tighten it, and don't tighten the holder nut without file in the hold. Don't mix-install $\Phi 0.7\text{mm}$ and $\Phi 0.8\text{mm}$ file on the file holder, the $\Phi 0.8\text{mm}$ file will not be able to be installed after a $\Phi 0.7\text{mm}$ installed.

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